

LMA | Classic™

The First LMA™ Airway



Mask Size	Patient Size	*Maximum Cuff Volume (Air)	Largest ETT ID (mm)
1	Neonates/infants up to 5 kg	up to 4 ml	3.5
1½	Infants 5-10 kg	up to 7 ml	4.0
2	Infants/children 10-20 kg	up to 10 ml	4.5
2½	Children 20-30 kg	up to 14 ml	5.0
3	Children 30-50 kg	up to 20 ml	6.0**
4	Adults 50-70 kg	up to 30 ml	6.0**
5	Adults 70-100 kg	up to 40 ml	7.0**
6	Large adults over 100 kg	up to 50 ml	7.0**

LMA | Unique™

The First Single-use LMA™ Airway



Mask Size	Patient Size	*Maximum Cuff Volume (Air)	Largest ETT ID (mm)
1	Neonates/infants up to 5 kg	up to 4 ml	3.5
1½	Infants 5-10 kg	up to 7 ml	4.0
2	Infants/children 10-20 kg	up to 10 ml	4.5
2½	Children 20-30 kg	up to 14 ml	5.0
3	Children 30-50 kg	up to 20 ml	6.0**
4	Adults 50-70 kg	up to 30 ml	6.0**
5	Adults 70-100 kg	up to 40 ml	7.0**

LMA | ProSeal™

The Most Versatile LMA™ Airway



Mask Size	Patient Size	*Maximum Cuff Volume (Air)	Largest Size OG Tube/Salem Sump	Largest ETT ID (mm)
1½	Infants 5-10 kg	up to 7 ml	10 French/8 French	4.0
2	Infants/children (10-20 kg)	up to 10 ml	10 French/8 French	4.5
2½	Children 20-30 kg	up to 14 ml	14 French/12 French	5.0
3	Children 30-50 kg	up to 20 ml	16 French/14 French	5.0
4	Adults 50-70 kg	up to 30 ml	16 French/14 French	5.0
5	Adults 70-100 kg	up to 40 ml	18 French/16 French	6.0**

*These are maximum volumes that should never be exceeded. It is recommended the cuff be inflated to 60 cm H₂O intracuff pressure.

** = cuffed ID = internal diameter ETT = endotracheal tube OG = orogastric

Never overinflate the cuff!

To place an order, please call (800) 788-7999

LATEX-FREE

LMA™ Quick Reference

LMA | Fastrach™ The Intubating LMA™ Airway



Mask Size	Patient Size	*Maximum Cuff Volume (Air)
3	Children 30-50 kg	up to 20 ml
4	Adults 50-70 kg	up to 30 ml
5	Adults 70-100 kg	up to 40 ml

LMA Fastrach™ Endotracheal Tube Sizes				
6.0 mm	6.5 mm	7.0 mm	7.5 mm	8.0 mm



LMA | Flexible™ / LMA | Flexible™ The LMA™ Airway for Head and Neck Cases

SINGLE USE



Mask Size	Patient Size	*Maximum Cuff Volume (Air)	Internal Diameter (mm)
2**	Infants/children 10-20 kg	up to 10 ml	5.1
2½	Children 20-30 kg	up to 14 ml	6.1
3	Children 30-50 kg	up to 20 ml	7.6
4	Adults 50-70 kg	up to 30 ml	7.6
5	Adults 70-100 kg	up to 40 ml	8.7
6***	Large adults over 100 kg	up to 50 ml	8.7

* These are maximum volumes that should never be exceeded. It is recommended the cuff be inflated to 60 cm H₂O intracuff pressure.

** Coming soon in single use.

*** Available in reusable mask only.

Never overinflate the cuff!

Single Use

Use Only 40 Times

Consult the instructions for information on indications, contraindications, warnings, and precautions.

LMA, LMA Classic, LMA ProSeal, LMA Fastrach, LMA Flexible, LMA Flexible Single Use and LMA Unique are trademarks of The Laryngeal Mask Company Limited.



LMA™

LMA North America Inc.

Putting Confidence in Your Hands™

(800) 788-7999

www.LMANA.com

LMA-145 09/04